


Reconfigure CMOS SPC 3365

When All Cables are fine and Message NO TRIGGER in RETIport when BIOSignal


Step 1:

Connect USB Keyboard via USB Adapter see image

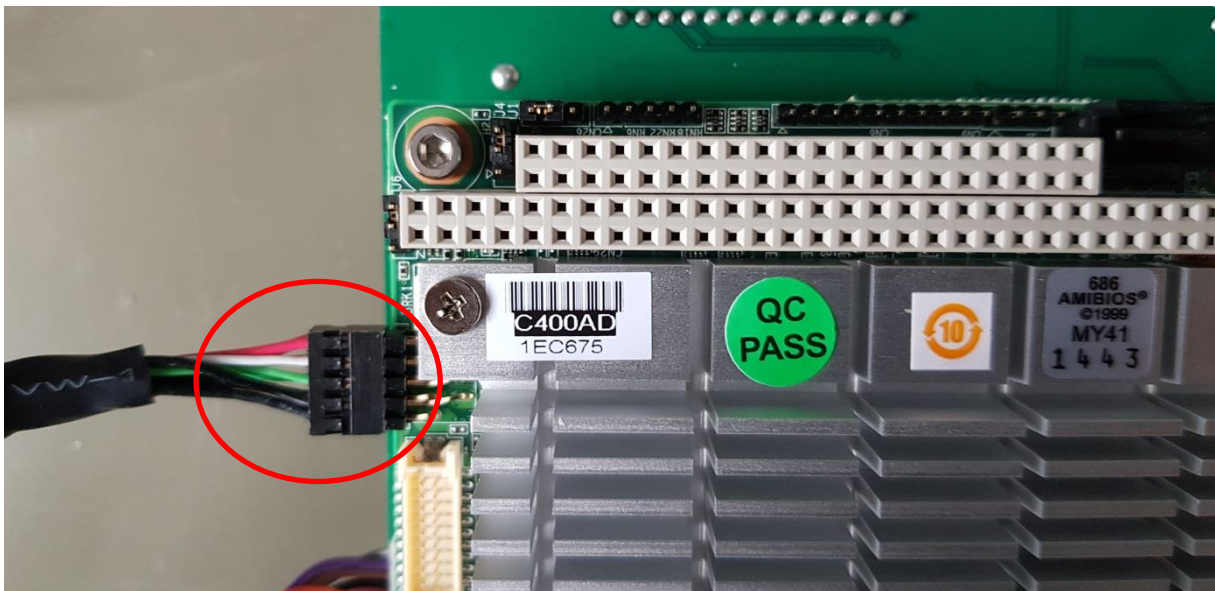

Step 2:

Clear CMOS with Jumper see in image

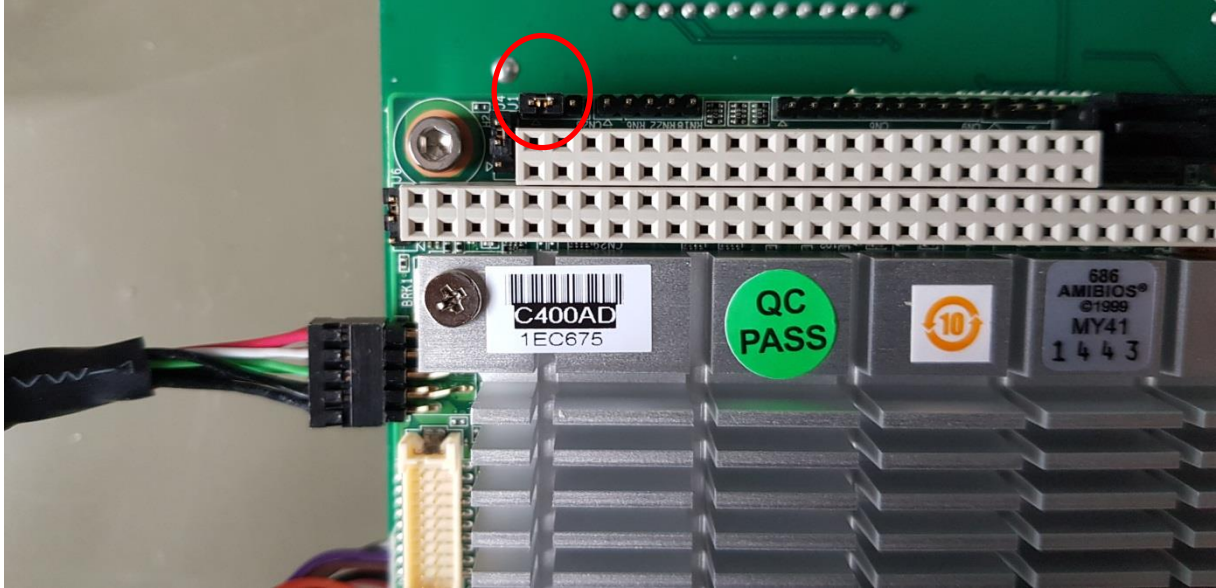

[--]* normal

*[--] clear

Step 3:



Start SPC with PC Power Connection

Press [DEL] to enter BIOS

Step 4:

Navigate to (Save & Exit) (Restore Defaults)

[ENTER]

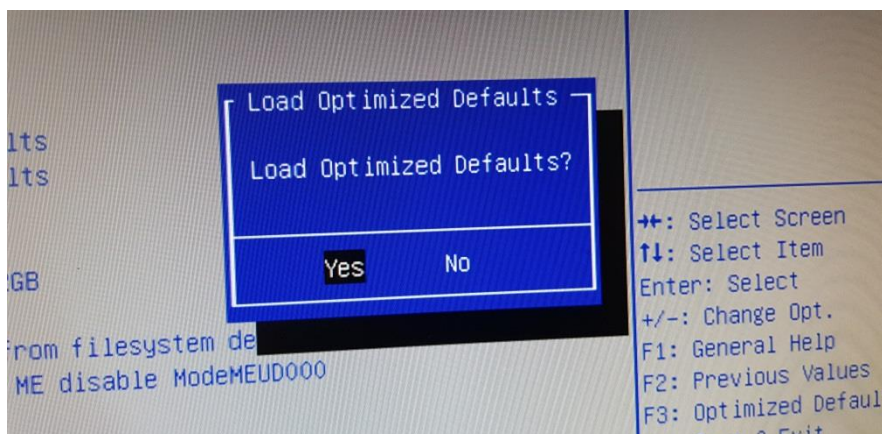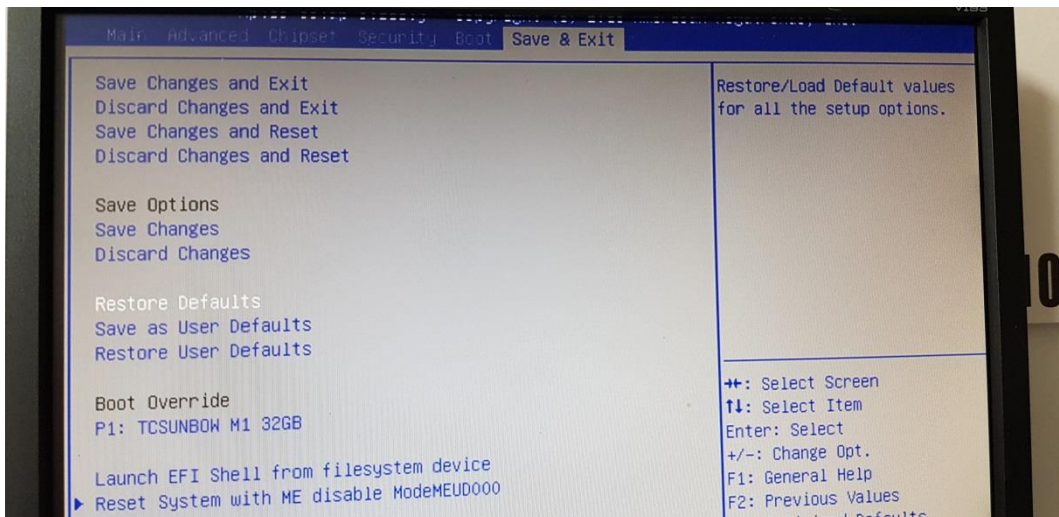



(YES)

[ENTER]

Step 5:

Navigate to (Save & Exit) (Save Changes and Exit)

[ENTER]

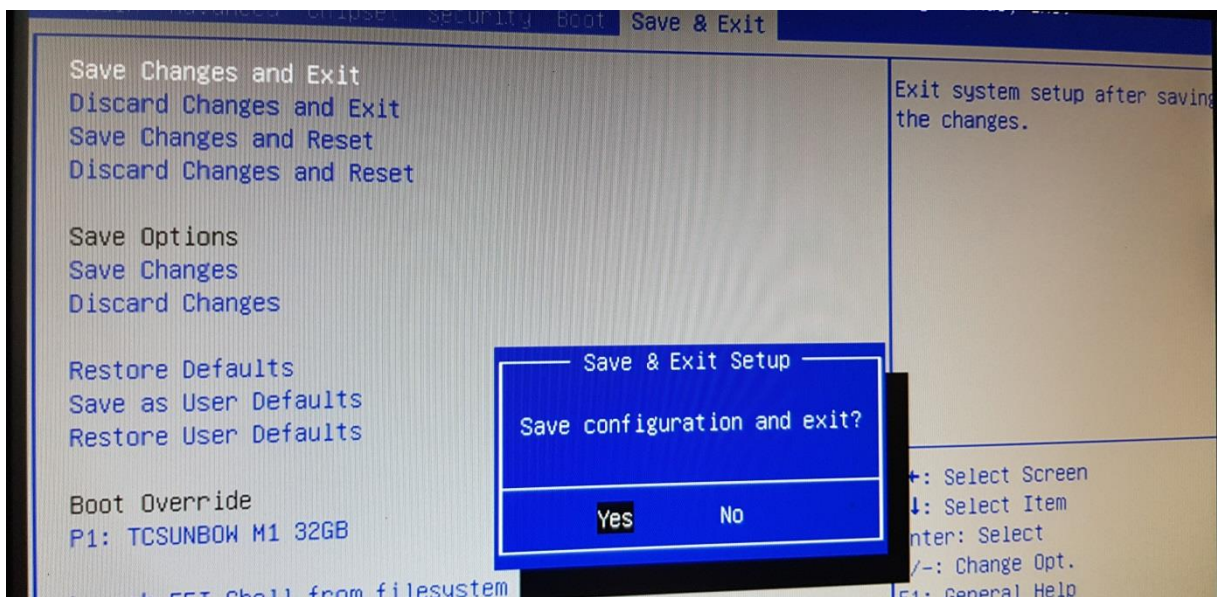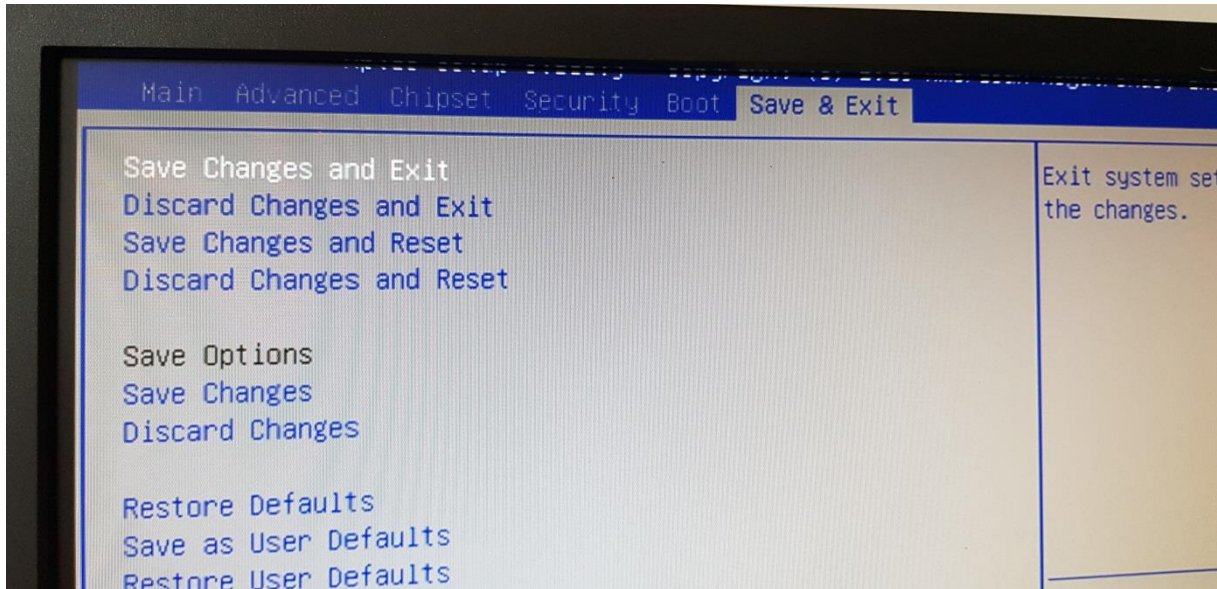

(YES)

[ENTER]

Step 6: REBOOT

Step 7:


Start SPC with PC Power Connection

Press [DEL] to enter BIOS

Step 8:

Navigate to (Advanced) (SCH3114 Super IO Configuration)


[ENTER]

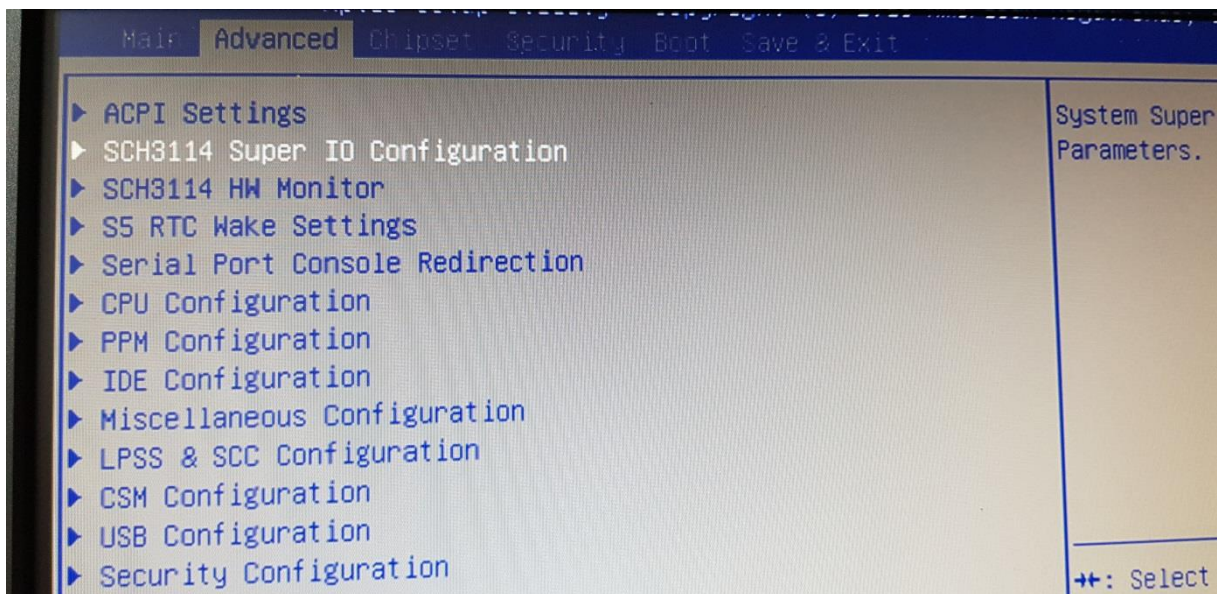

Step 9:

Navigate to (Serial Port 1 Configuration)



[ENTER]

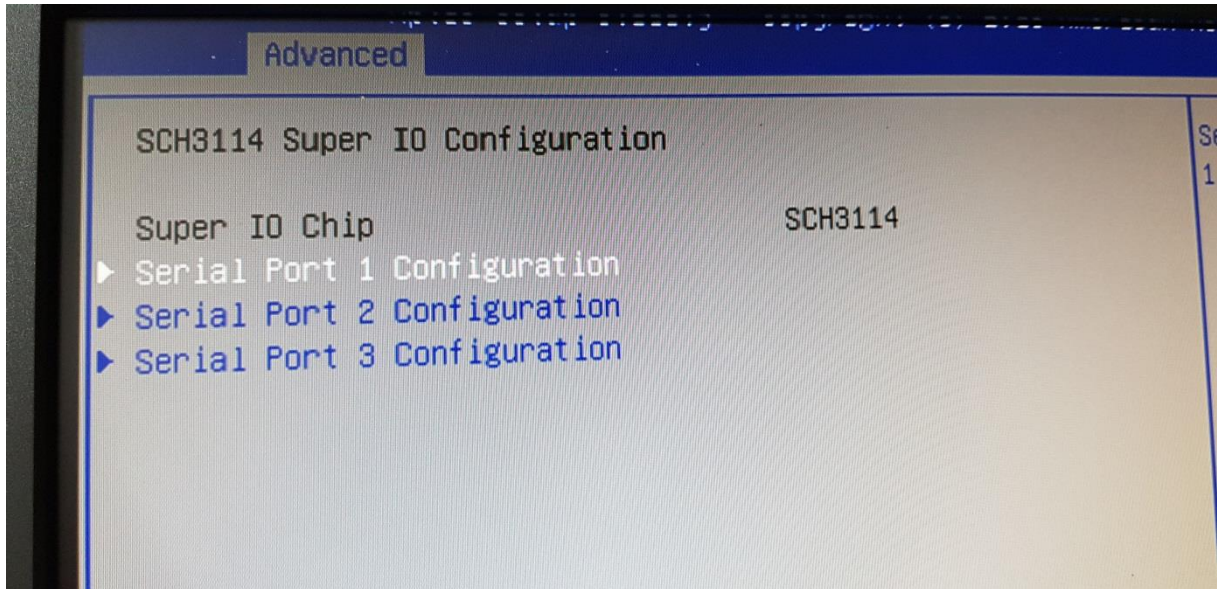

Step 10:


Navigate to (Change Settings)

[ENTER]

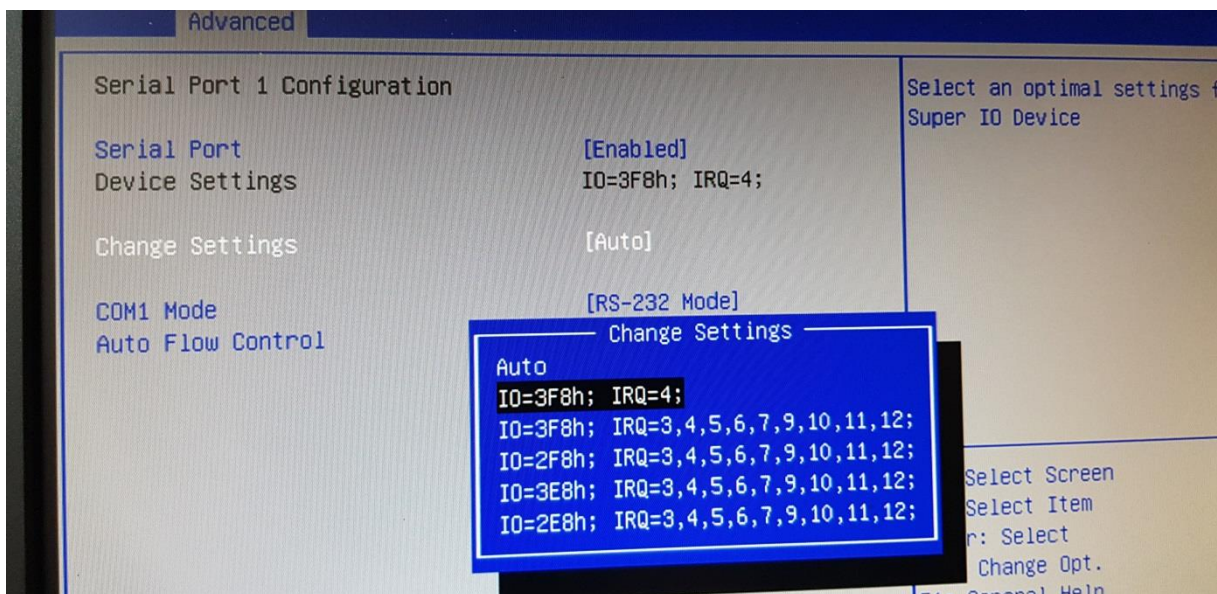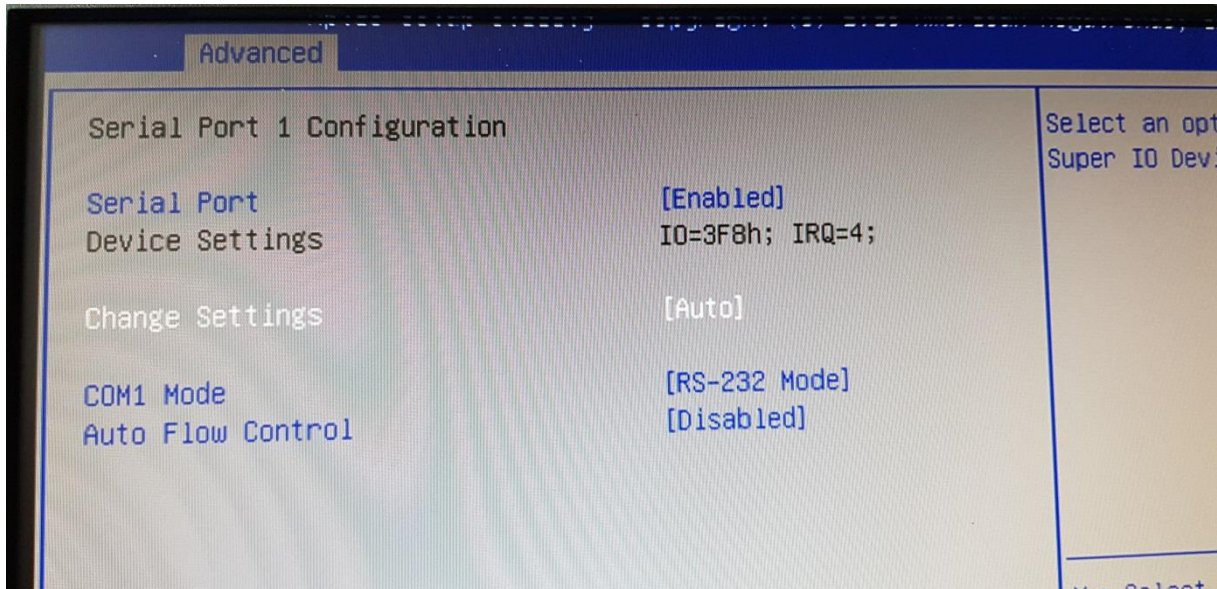

From (Auto) to (IO=3F8h)


[ENTER]

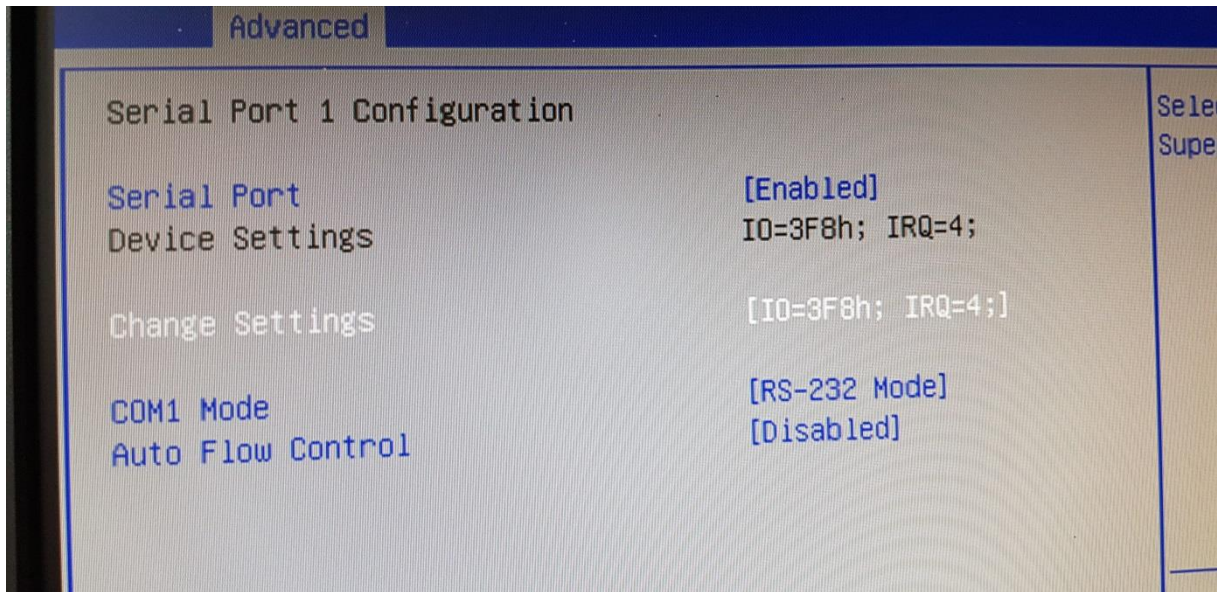

Step 11:

Navigate to (Serial Port 2 Configuration)



[ENTER]

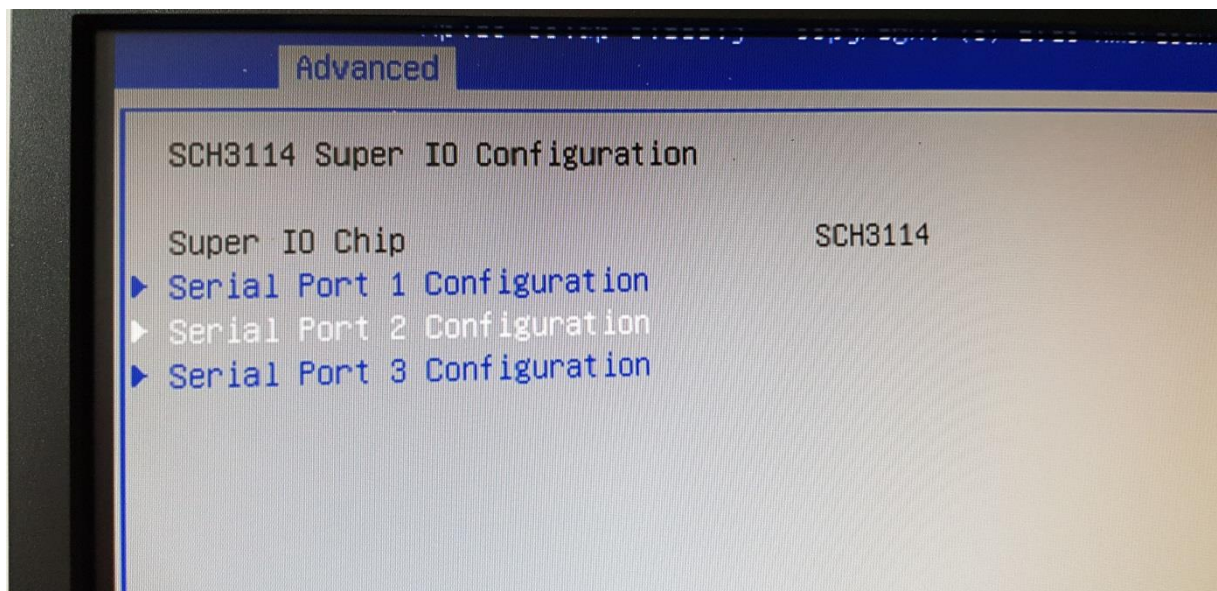

Step 12:


Navigate to (Change Settings)

[ENTER]

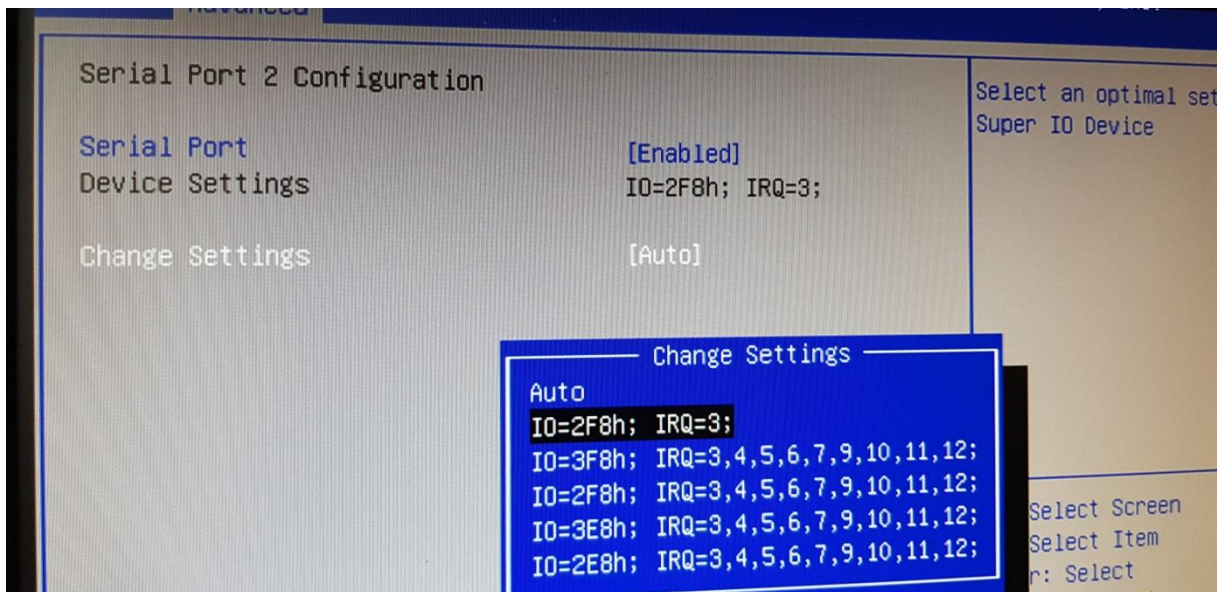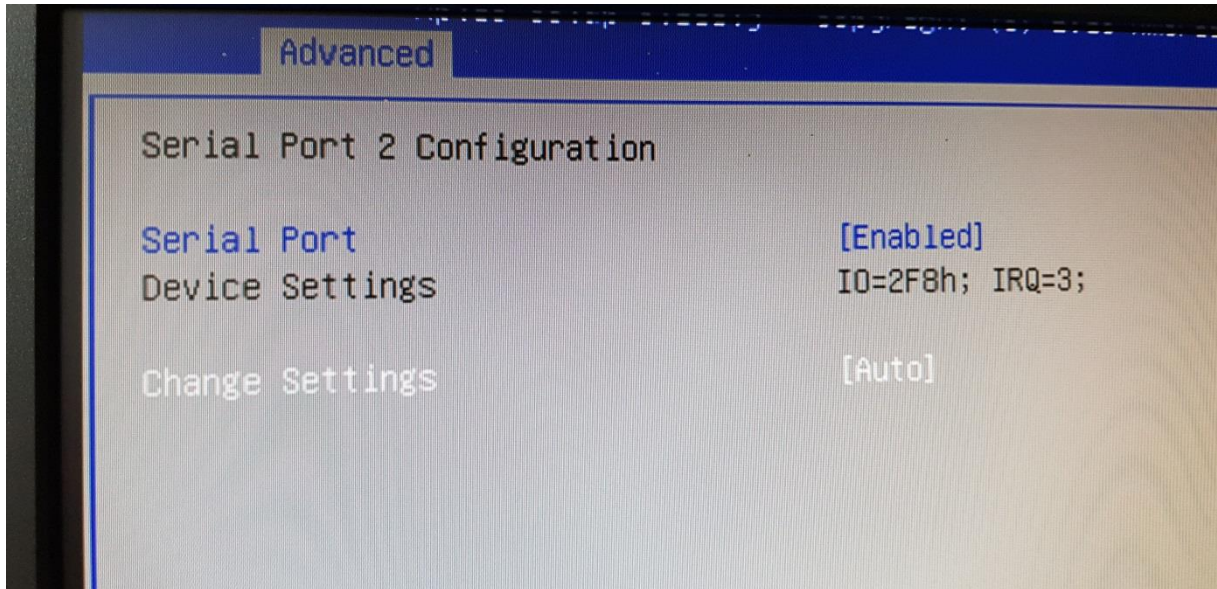

From (Auto) to (IO=2F8h)


[ENTER]

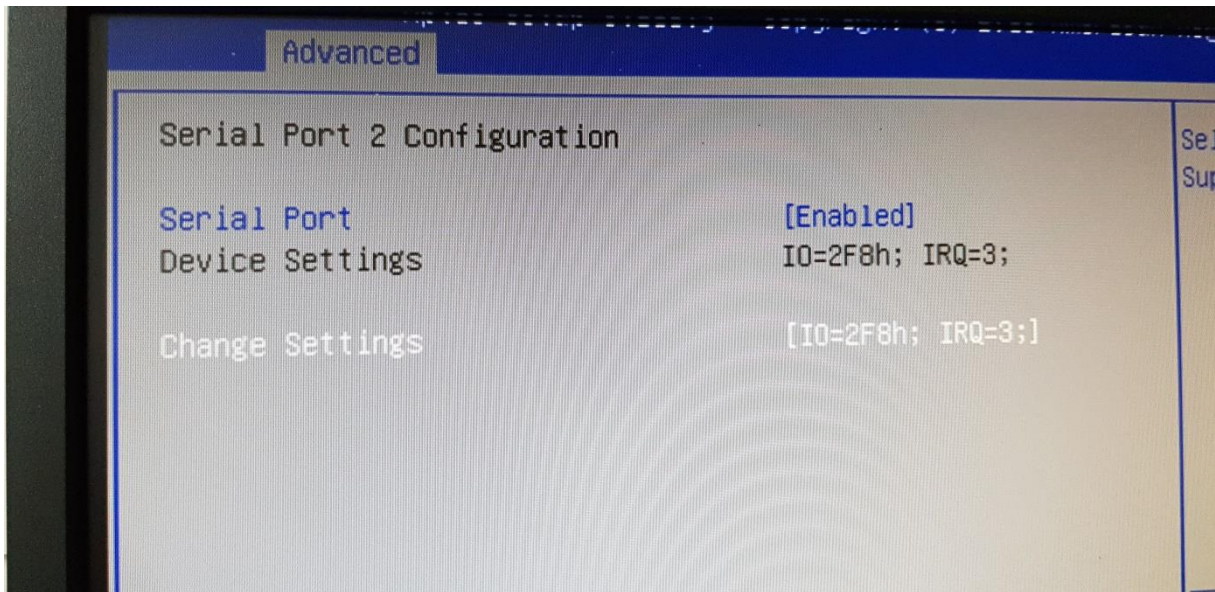

Step 13:

Navigate to (Serial Port 3 Configuration)



[ENTER]

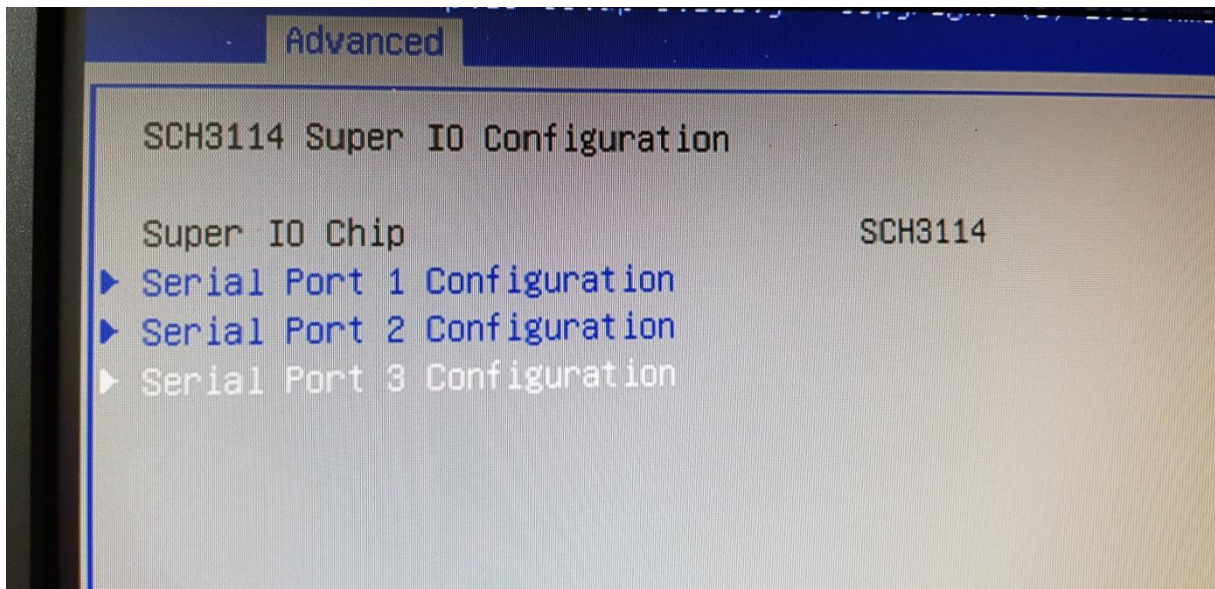

Step 14:

Navigate to (Change Settings)

[ENTER]

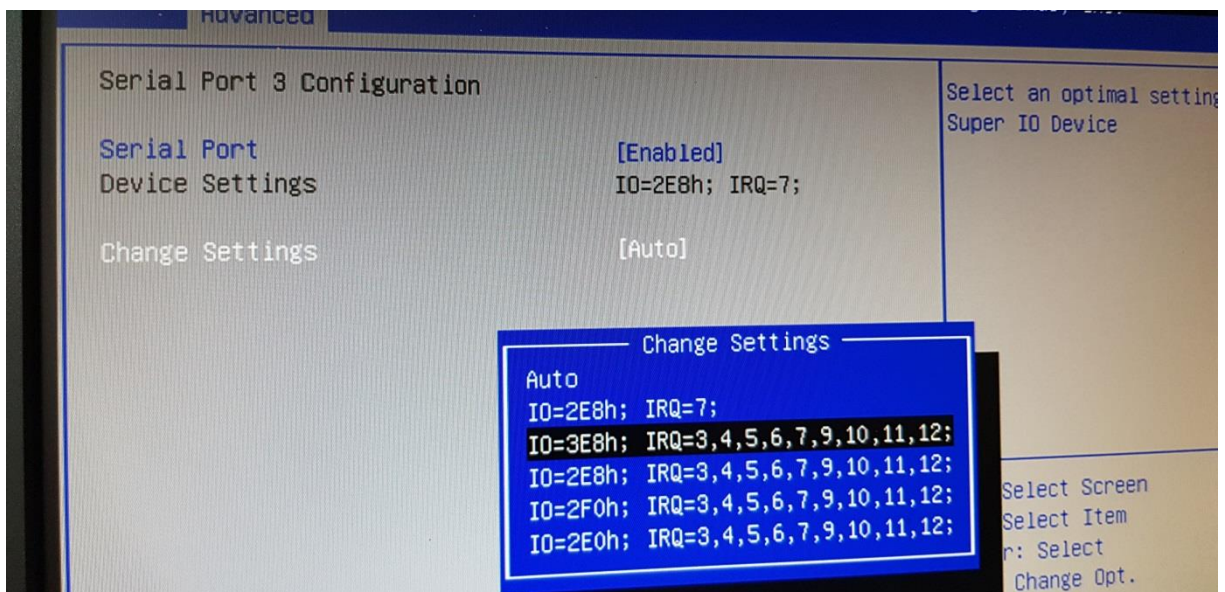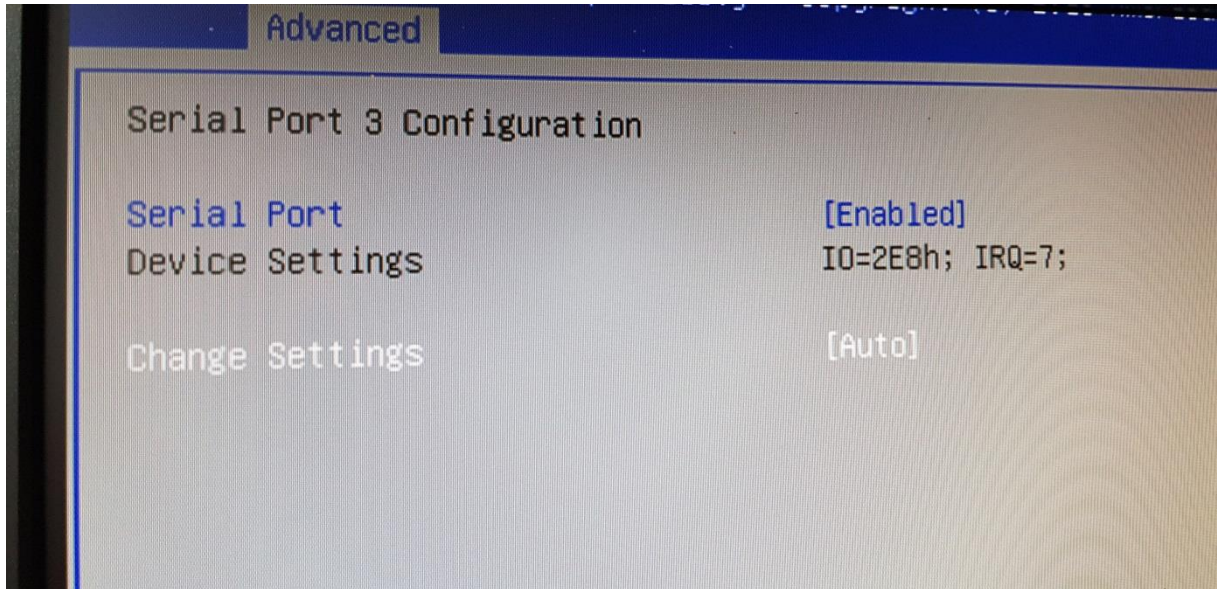

From (Auto) to (IO=3E8h)

[ENTER]

Advanced

Serial Port 3 Configuration

Serial Port	[Enabled]
Device Settings	IO=2E8h; IRQ=7;
Change Settings	[IO=3E8h; IRQ=3,4,5,...]



Select an
Super IO D

++: Select

Step 15:

Navigate to (Save & Exit) (Save Changes and Exit)

[ENTER]

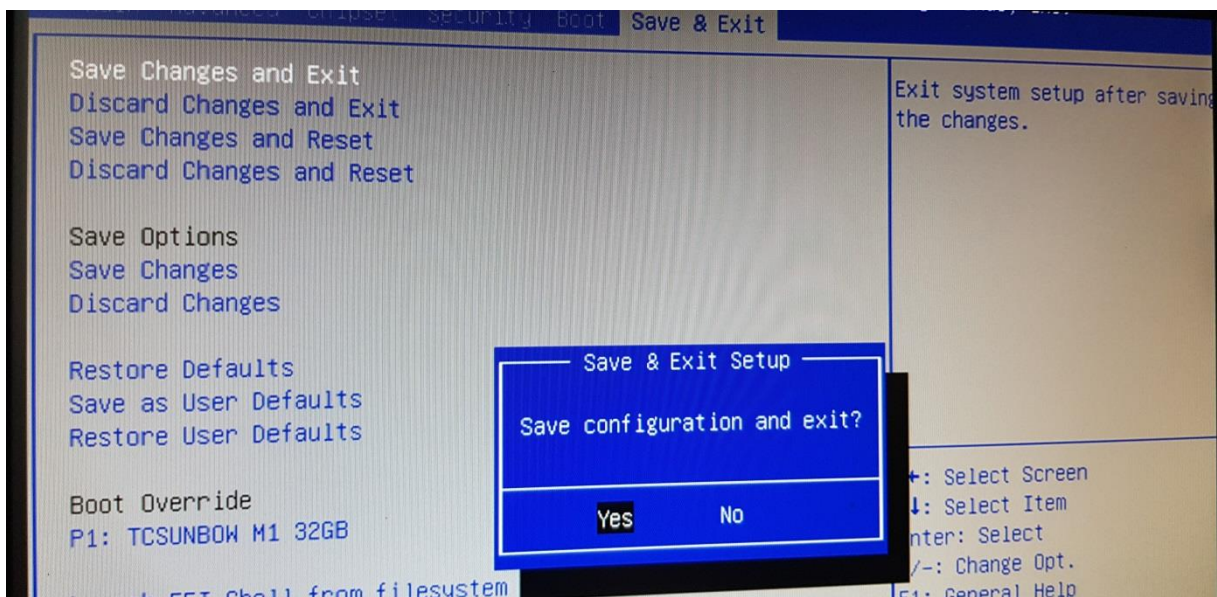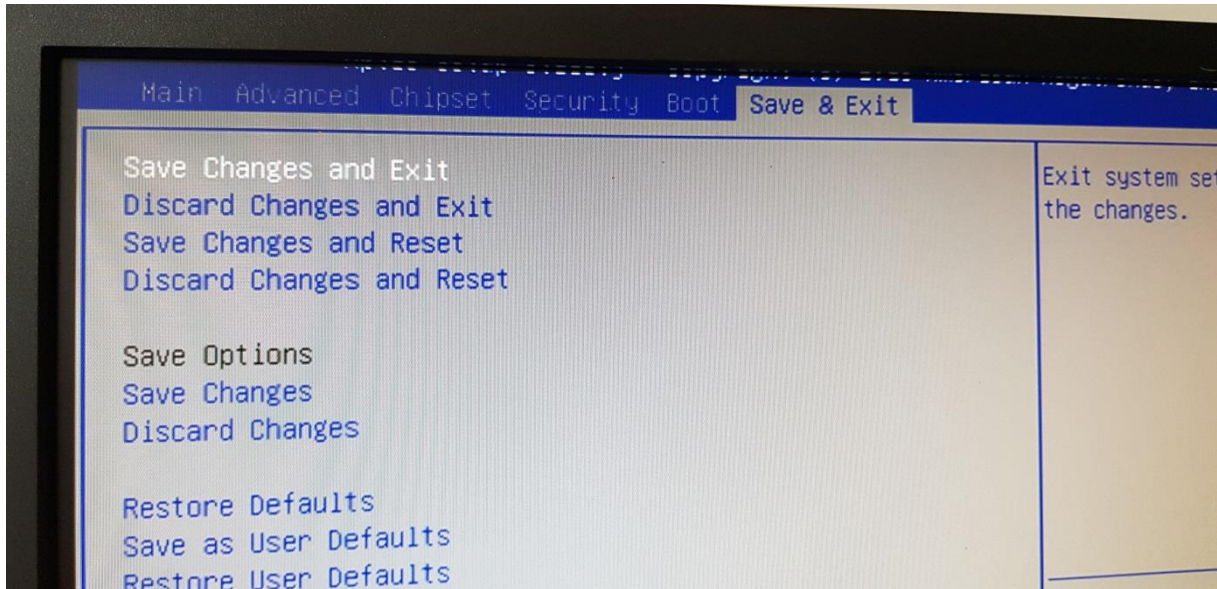

(YES)

[ENTER]

Step 16: REBOOT

Step 17:


Start SPC with PC Power Connection

Press [DEL] to enter BIOS

Step 18:

Navigate to (Advanced) (SCH3114 Super IO Configuration)


[ENTER]

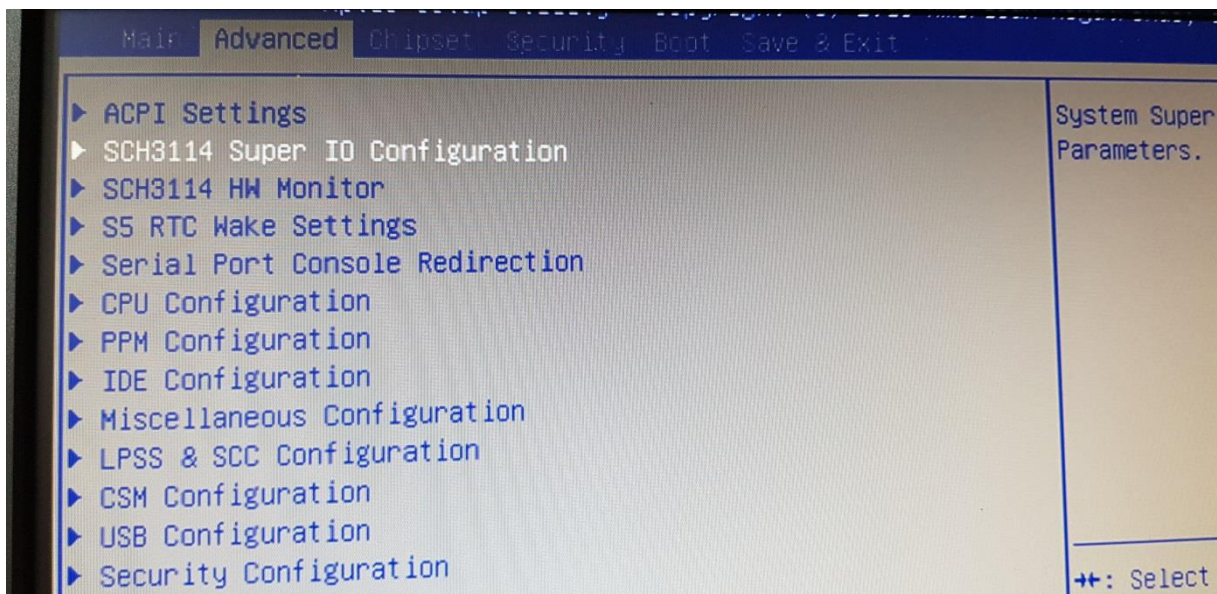

Step 19:


Navigate to (Serial Port 1 Configuration)

[ENTER]

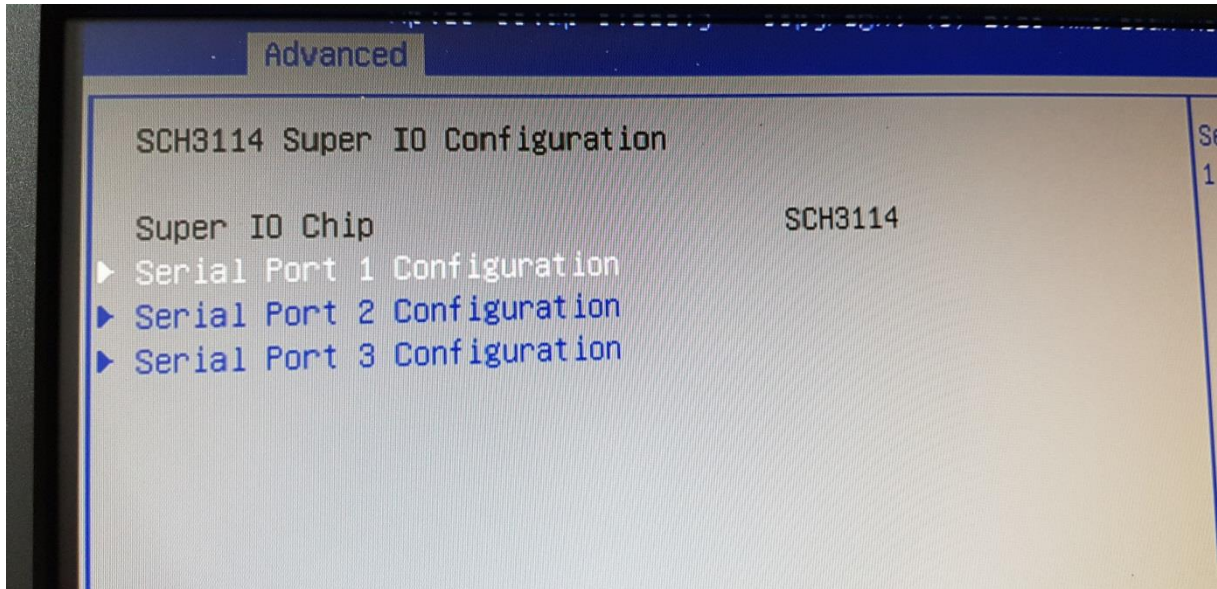

Step 20:

Check if Setting is **not** (AUTO)

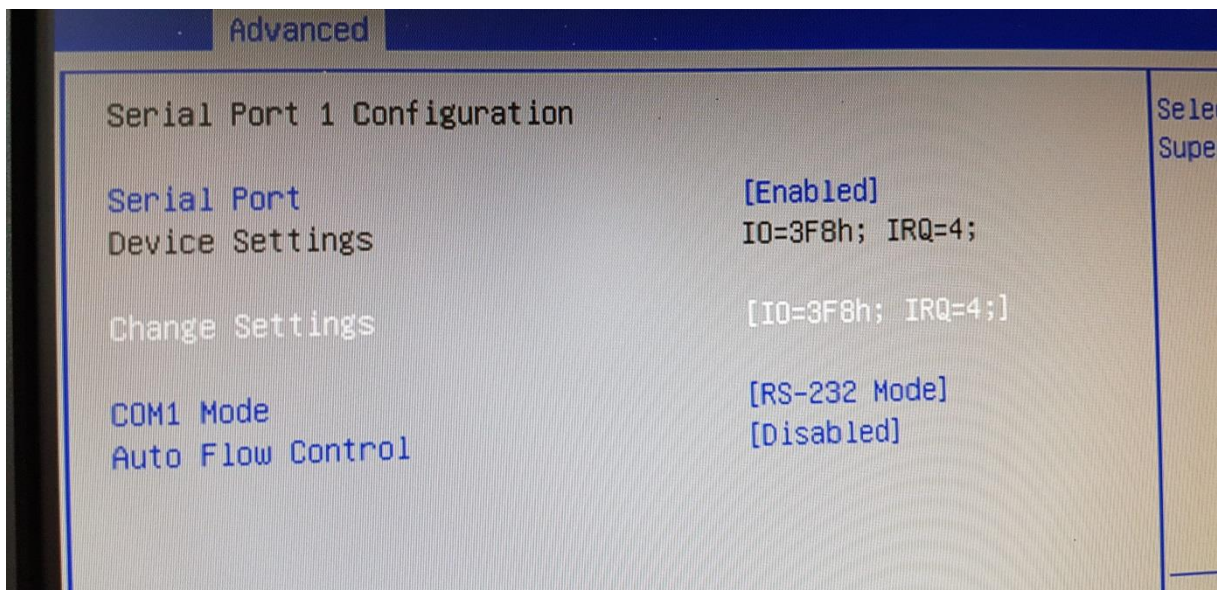

Step 21:


IF Setting is **(AUTO)** Repeat from **Step 8 !!!**

IF Setting is **([IO=3F8...])** goto **Step 22:**

Step 22:

Navigate to (Chipset)(South Bridge)


[ENTER]

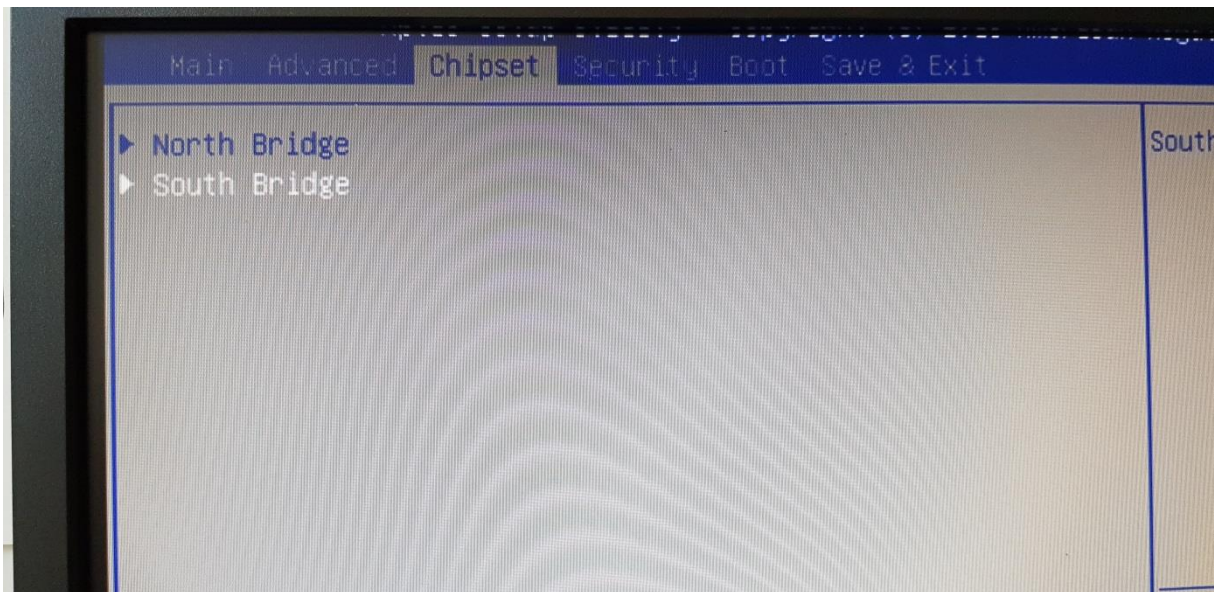

Step 23:

Navigate to (USB Configuration)



[ENTER]

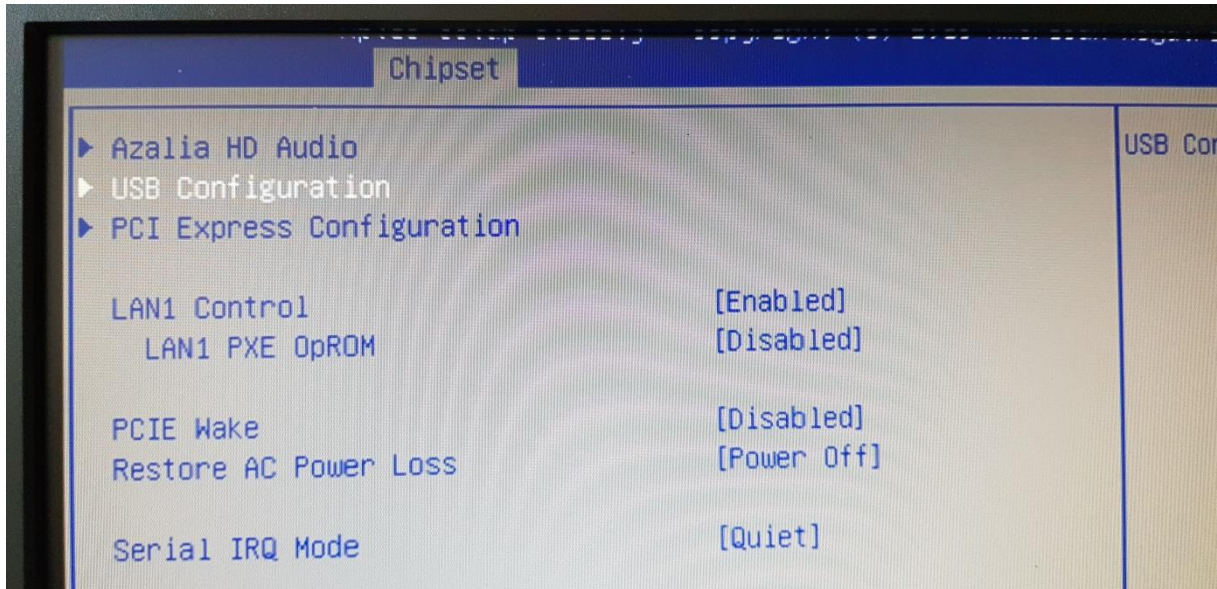

Step 24:

Navigate to (USB Configuration)

[ENTER]

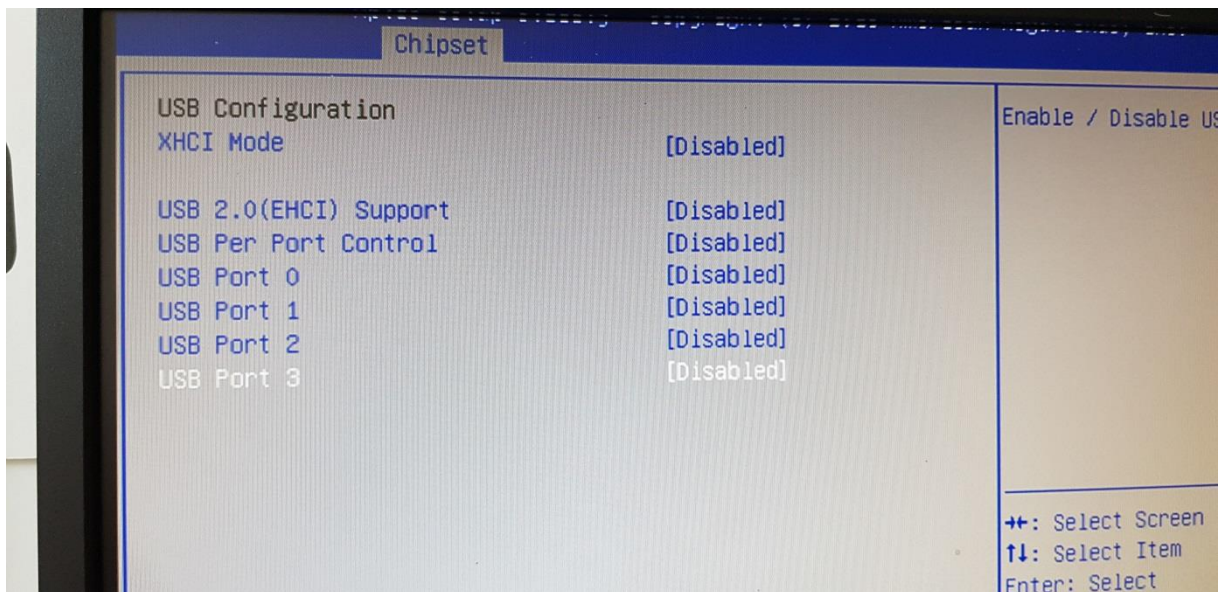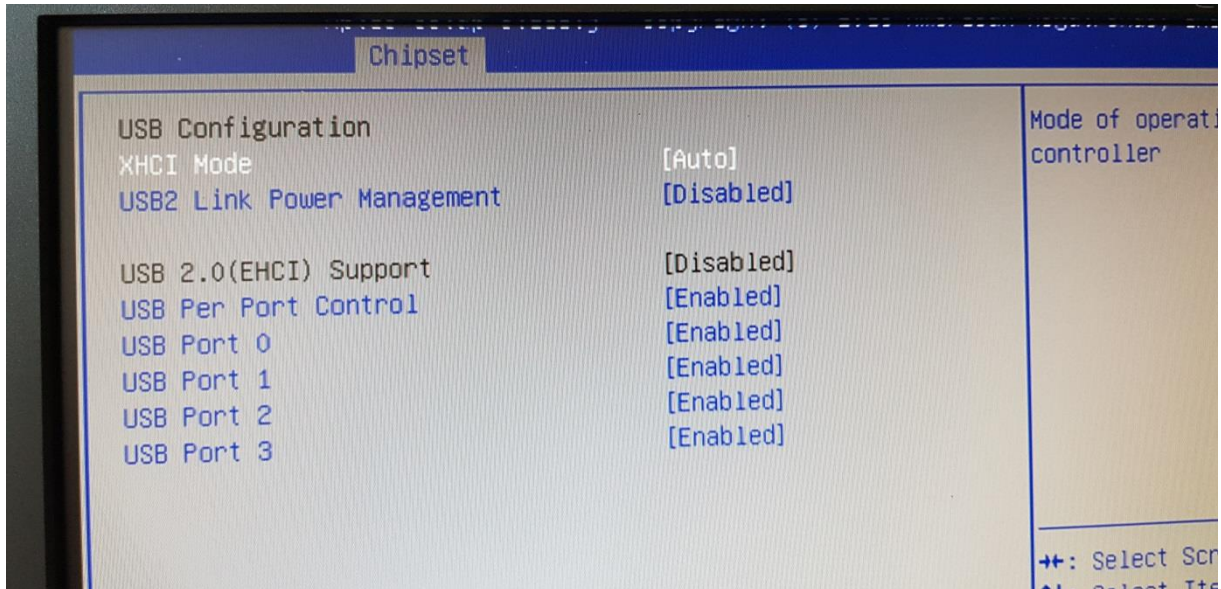


Set all to [Disabled]



Step 25:

Navigate to (Advanced)(USB Configuration)

[ENTER]

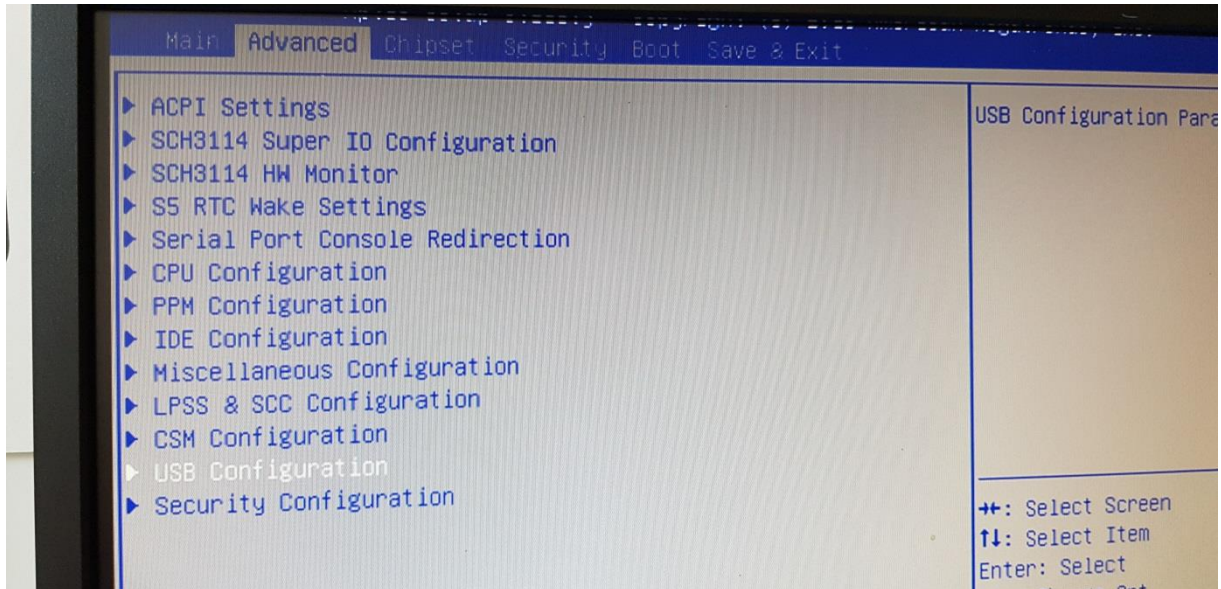

Step 26:

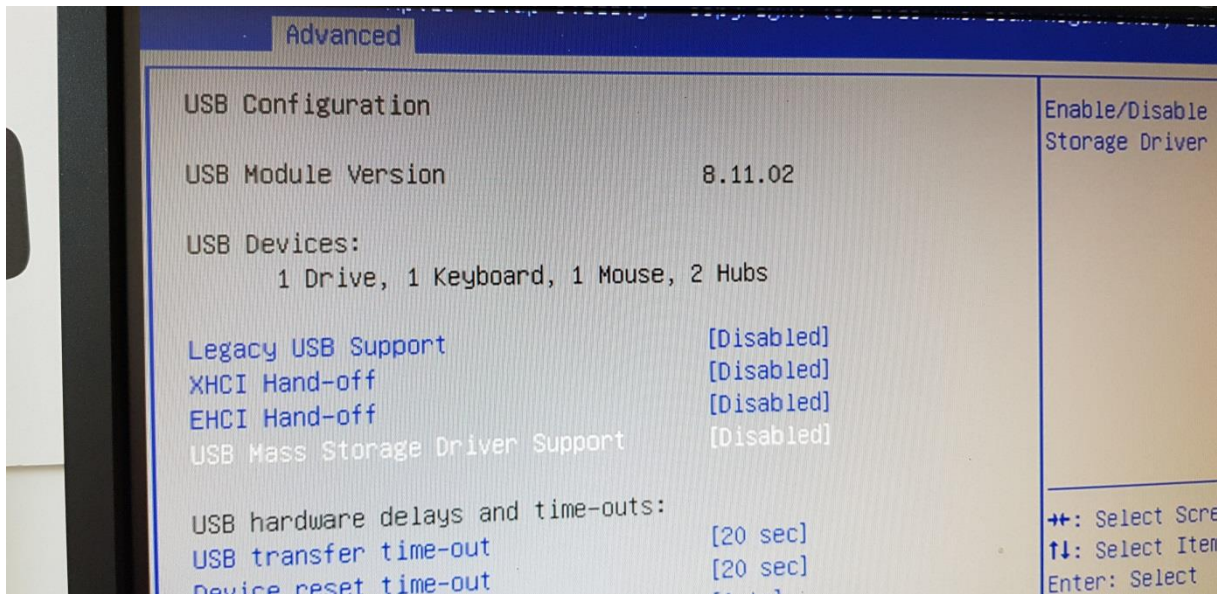
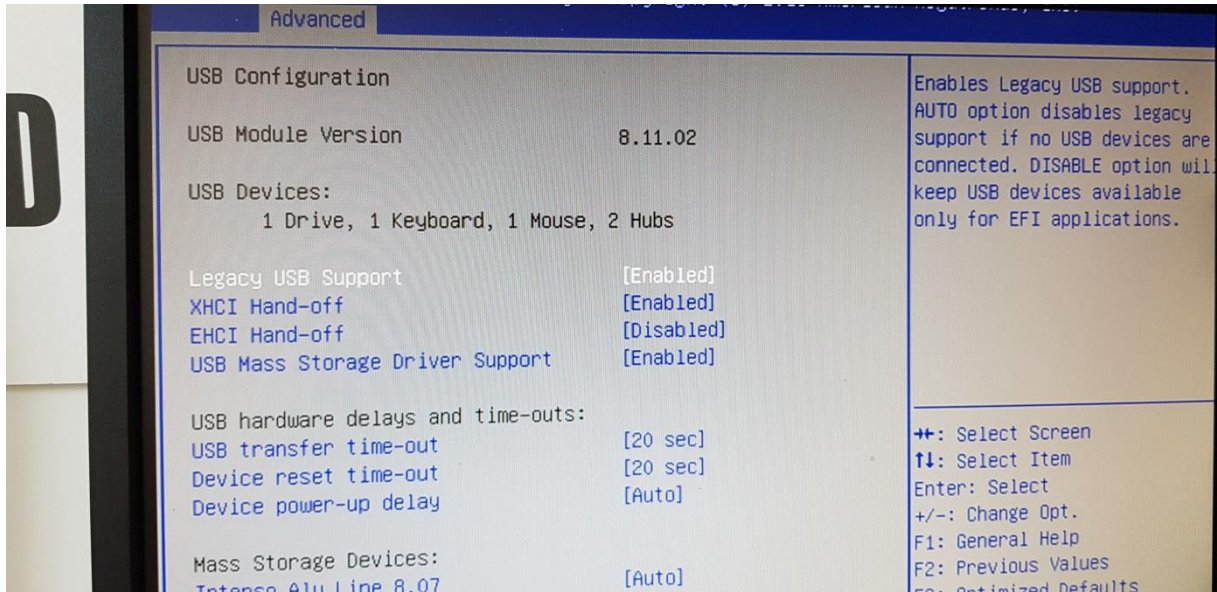


Set **these** to [Disabled]

Step 27:

Navigate to (Save & Exit) (Save Changes and Exit)

[ENTER]

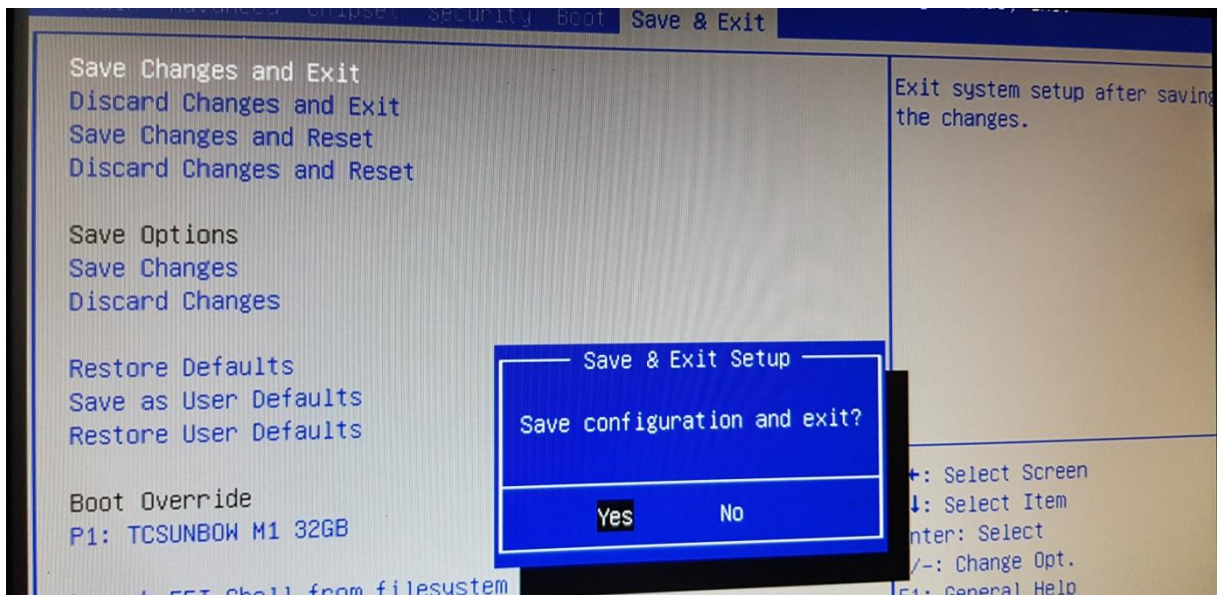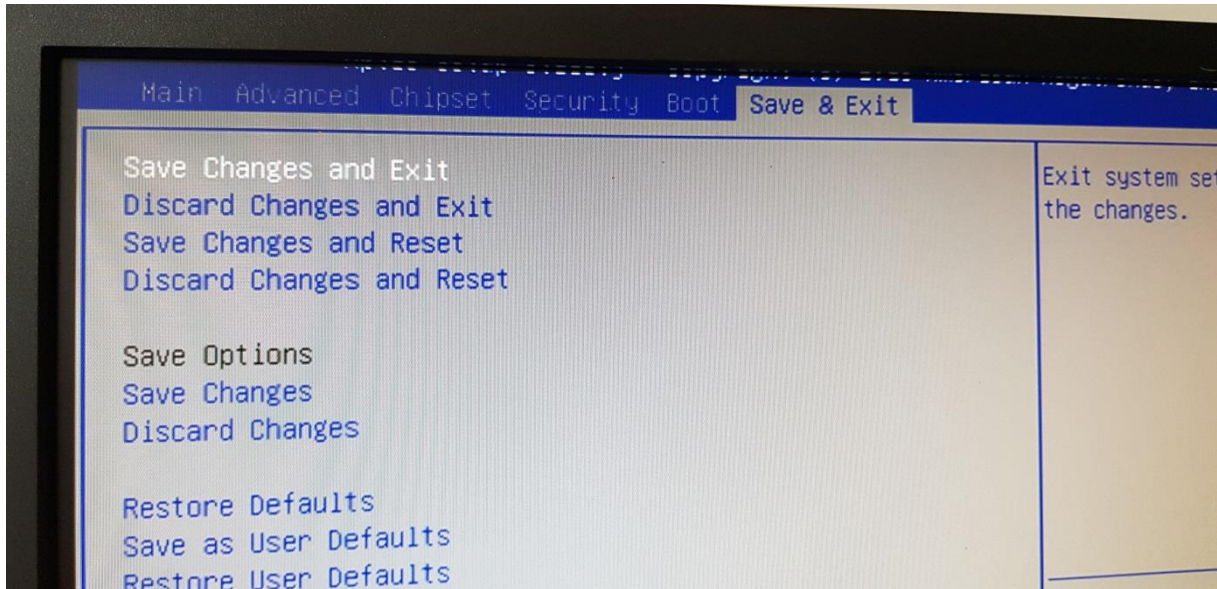

(YES)

[ENTER]

Step 28:

Test with **RETIport** if Message **No TRIGGER** is gone!