

## RCDB c/s Installation

### A) Root directory at server

- A.1. define the server DB location "DB root" (e.g. the default d:\reti10\RCDB) and create the directory if no current database is set up: If the mode shall be changed from file based to C/S operation the current path can be found in reticonf.ini:  
[rcdb10]  
BasePath=

### B) Server install:

- B.1. copy the server directory to this RCDB directory
- B.2. copy the server ini "edbsvr.ini" to  
C:\ProgramData\Elevate Software\ElevateDB Server\
- B.3. default server port is 12010, may be changed in the edbsvr.ini and in the client config if the port is already in use
- B.4. create firewall rule to enable tcp port (12010) at the server
- B.5. run command shell with admin privileges and navigate to "DB root"\server
- B.6. install service "edbsvr.exe /install"  
(service will be installed with name "RC database server")
- B.7. run "services" app and start service (should be set to autostart)

### C) Network share

- C.1. Server: create network share with r/w permission for "DB root" directory
- C.2. Client: map the network share with auto-reconnect to the same drive letter on every client (e.g. "R:\")

### D) reticonf.ini settings

Configure the DB entries in reticonf.ini rcdb10 section [RCDB10] at every station

- D.1. set "host=" to localhost or 127.0.0.1 at the machine running the DB service  
(NOT the own IP!)  
set "host=(server ip address) on all client stations
- D.2. set "port=" to the choosen port (2)
- D.3. set "ServerBasePath=" to the local "DB root" path at the server (1)

D.4. set "MappedBasePath=" to the mapped network share (11)

The "BasePath=" entry is unused in c/s mode and will be set to ServerBasePath value.

#### **E) If Database path changes**

If the DB root folder changes (moving the DB to another location or convert a local DB to c/s mode), the path entries in the databases and the backup store must be adjusted. **Done by RC service staff.**

We suggest do leave the DB path the same at the Server when switching from file based to C/S mode!

#### **Local file based mode:**

Just set the "host=" entry to empty and set the "BasePath=" to the DB root directory. Empty Base path leads in creating DB structure below the reti installation directory (default "d:\reti10") in the "RCDB10" subdirectory.