



Fix for USB Cameras with Mainboard AS Rock B85 Pro4



1. Go to Bios (F2 while booting)
 - 1.1 Click Advanced
 - 1.2 Click USB Configuration

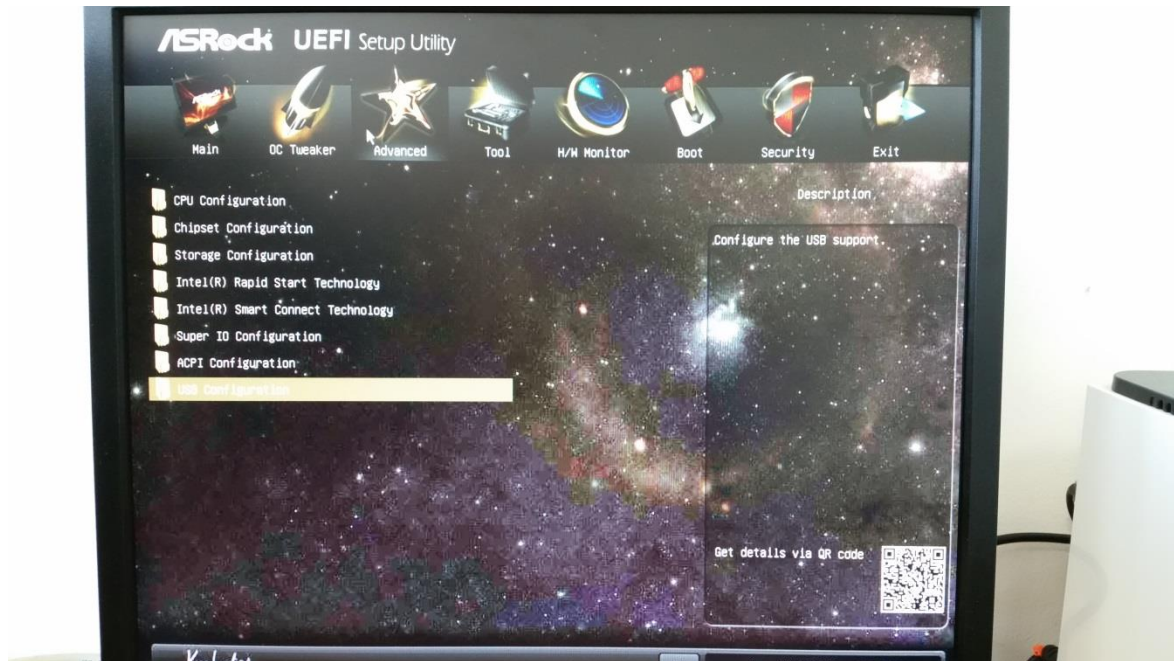

2. USB Configuration Page
 - 2.1 USB Controller = Enabled
 - 2.2 Inter USB3.0 Mode = Disabled
 - 2.3 Legacy USB Support = Auto
 - 2.4 USB Compatibility Patch = Disabled

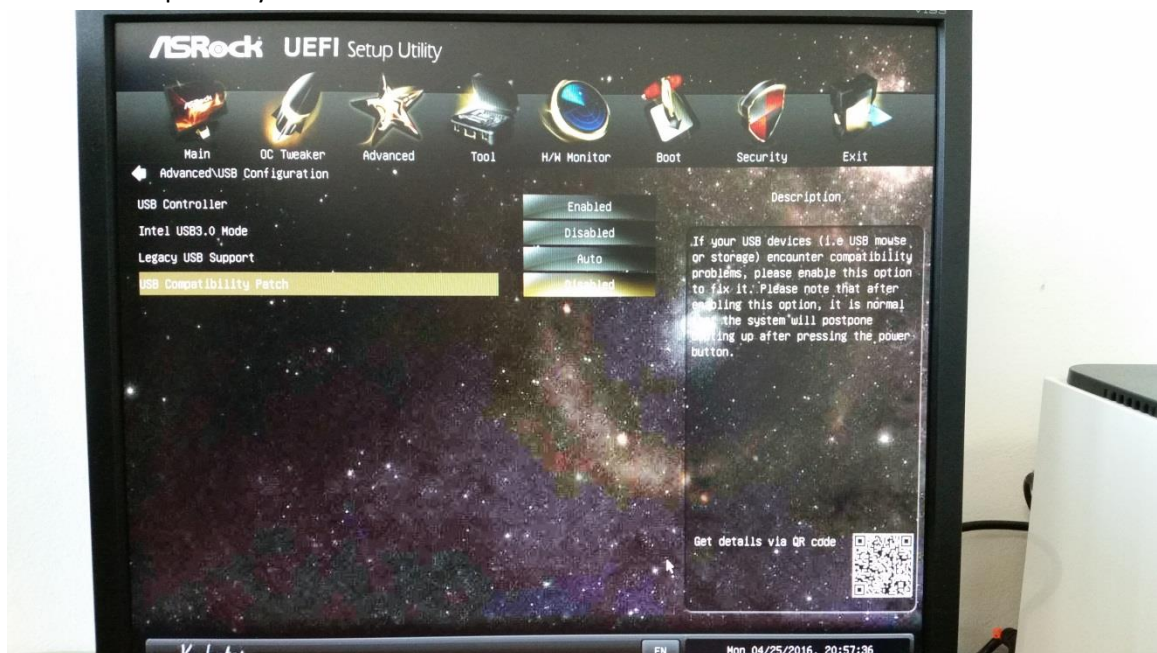

3. Save Bios Settings

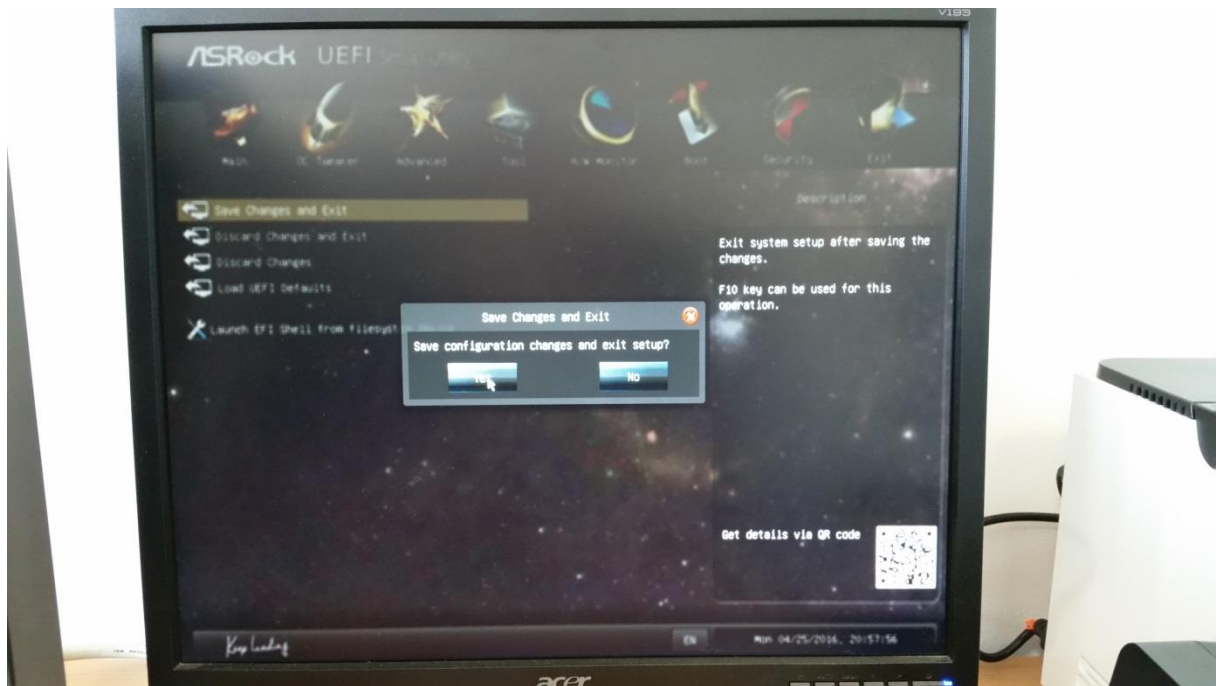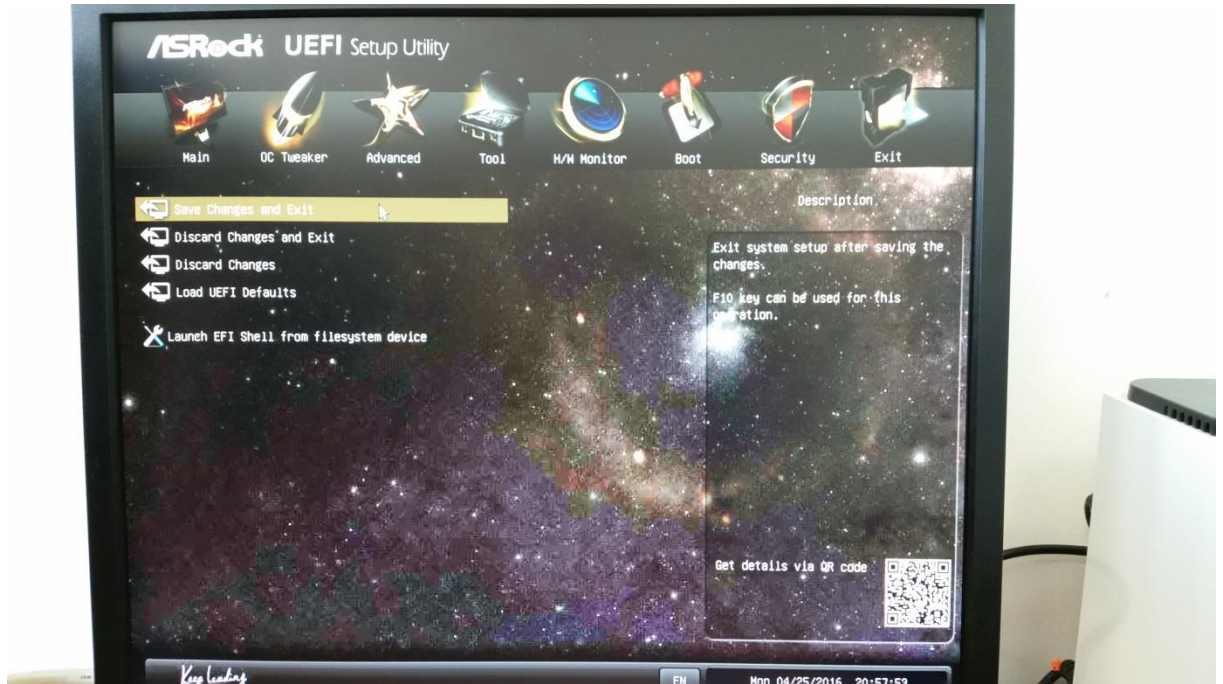

4. Start Windows, USB Driver will reinstalling self
5. Start RETImenu
 - 5.1 Click Service
 - 5.2 Click Configuration
 - 5.3 Click Cameras
 - 5.4 Check "IR Camera" if it is off
 - 5.5 Rightclick "IR CAM" Frame
 - 5.6 Click Configuration
 - 5.7 Set the Cameras and Click OK
 - 5.8 Rightclick "IR CAM" Frame
 - 5.9 Move Mouse to Camera -> Select the Cameras "Monitor" and so on
 - 5.10 The Cameraimage should be displayed (or Bluescreen or Freeze)
 - 5.11 If Bluescreen or Freeze repeat procedure 4. and 5. until it works. (~3 times max)

